
SnoreO2

Snore Reducing Sleep Monitor

User Manual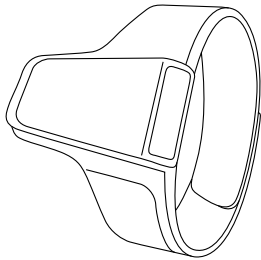

www.viatomtech.com

Overview

Snoring may cause breathing problem reducing oxygen level in body. SnoreO2 monitors oxygen level, heart rate and motion during sleep.

SnoreO2 stirs your body by smart vibration in the event of hypoxia to relieve oxygen shortage. Vibration will abort automatically if oxygen condition turn well.


SnoreO2 provides O2 Score for long-term tracking and analysis, helps you stay away unhealthy lifestyles.

This product is not a medical device and should not be used to diagnose or treat any medical condition.

DO NOT squeeze the sensor or apply excessive force on the sensor & cable.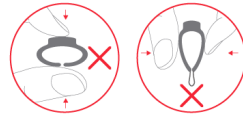

Package list

(1) Main Unit; (2) Soft Thumb Ring Sensor; (3) USB charging Cable; (4) User Manual

Charging

Please charge the battery before the first use.

Connect the device to the USB port of personal computer (PC) using the USB Charging Cable; typically takes less than 2 hours to fully charged. A full charge can be used for at least two nights.

Power ON/OFF

Press and hold the button on the side for 1 second to power ON, 3 seconds to power OFF. It turns off automatically if no measurement or operation.

Download SnoreO2 App

App name: SnoreO2

iOS: App Store; Android: Google Play

LED indication

Green Flash	Before monitoring/Fully charged
Green ON (turns off in 2 minutes if no operation)	Normal monitoring
Blue Flash	Monitoring finished

Blue ON	Bluetooth connected/Charging
Blue-Green staggered	Low battery/Fault

Start monitoring

1. Connect the thumb ring sensor to the port on device.
2. Put SnoreO2 on your wrist.
3. Slide the ring sensor onto your thumb as pictured.
4. Turn on the device, the LED indicator will turn green. Then it starts to monitor and record.

Stop monitoring

Take off the Ring Sensor, the monitoring and recording will finish when LED turns to blue flash in 10 seconds. Then you can sync data at this moment or later. It will power off automatically in minutes if no operation.

Establish Bluetooth connection

The device *Bluetooth* will open when LED is ON. To establish *Bluetooth* connection,

1. Power on device. If LED is off, press button to turn on the LED.
2. Enable the phone *Bluetooth* and run the SnoreO2 App.
3. Choose the right Device ID for the initial use.

Real-time View

When *Bluetooth* is connected, Oxygen Level , heart Rate and battery are shown in **Dashboard** in App in real-time.

Note: to get stable readings, avoid excess motion of your hand.

Smart Vibration

In App, you can set Smart Vibration to meet personal need. The threshold and strength of vibration can be adjusted.

Sync data

After a record is finished, you can sync data from the device to App.

Note: Your SnoreO2 can store a maximum of 2 records; the oldest record will be overwritten by new record. Please sync data in time.

History data and sleep report

In App, you can review all the history data.

Touch a recording, there comes out the detailed report. It includes analysis results and plots.

Slide a recording to left, you can delete it.

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